

Spectrums of activity and applications

	Disinfection	Cleaning	VAH listed	IHO virucide list	certified in acc. with EU standards	Bactericidal	Tuberculocidal	Yeasticidal	Fungicidal	Limited virucidal	Limited virucidal plus	Virucidal	Concentration / Reaction time in min.	No. of days product works for
Instruments														
ID 215 Enzymatic instrument cleaner	■	■			■	■		■		■			1%/2%/30/15	
Surfaces														
FD 322 Quick-acting disinfection	■	■	■	■	■	■	■	■	■	■	■		100%/1*	
FD 333 Quick-acting disinfection	■	■	■	■	■	■	■	■	■	■	■		100%/1*	
FD 333 top wipes Quick-acting disinfection	■	■	■	■	■	■	■	■	■	■	■			
FD 360 Cleaning and care of vinyl upholstery		■											ready to use /1*	
FD 366 sensitive wipes for disinfection of sensitive surfaces	■	■	■	■	■	■	■	■		■			ready to use /1*	
FD 366 sensitive top wipes for disinfection of sensitive surfaces	■	■	■	■	■	■	■	■		■			ready to use /1*	
Special areas														
MD 550 Spittoon bowl cleaner		■		■	■	■		■					100%*	
Orotol® plus Disinfection of suction systems	■	■	■	■	■	■		■		■			2%/5*	
MD 555 cleaner Special detergent for suction units		■											5%/30-120	

* bactericidal/yeasticidal.

For more information, please refer to the product information or visit www.duerrdental.com



DÜRR DENTAL GLOBAL GMBH
 PO Box 184
 Balaclava Victoria 3183
www.duerrdental.com/en/AU/
frank.schroeder@duerrdental.com

DÜRR DENTAL
 THE BEST, BY DESIGN

System Hygiene from Dürr Dental



■ ID 215 Enzymatic Cleaner

Concentrate for manual, enzymatic instrument cleaning prior to disinfection.



■ FD 322 Quick-acting disinfection

Fast-acting, ready-to-use solution for the disinfection and cleaning of surfaces. Recommended by Planmeca, KAVO & Heka.



■ FD 333 Quick-acting disinfection

Ready-to-use solution for the quick-acting disinfection of the surfaces of medical devices and medical inventory. Recommended by Planmeca, Morita, Belmont & Heka.



■ FD 333 top wipes Quick-acting disinfection

Alcohol-based, ready-to-use disinfection and cleaning wipes for smaller surfaces of medical devices. Recommended by Belmont, Morita, Planmeca & Heka.



■ FD 360

Cleaning and care of vinyl upholstery. Recommended by Planmeca, DentsplySirona, Morita & Heka.



■ FD 366 Sensitive disinfection of sensitive surfaces

Alcohol-reduced, ready-to-use solution for the disinfection and cleaning of especially sensitive surfaces. Recommended by A-dec, DentsplySirona, Morita, Belmont & Heka.



■ FD 366 Sensitive wipes and top wipes Disinfection of sensitive surfaces

Ready-to-use disinfection and cleaning wipes for sensitive surfaces. Recommended by A-dec & Dentsply Sirona.

Available as: Dispenser box with 100 wipes
Wipe size: 20 x 30 cm



■ MD 550 Spittoon bowl cleaner

Foam-free, ready-to-use solution for intensive, residue-free cleaning of spittoons. Recommended by Dentsply Sirona, Morita & Heka.



■ Orotol® plus Disinfection of suction systems

Foam-free liquid concentrate for the simultaneous disinfection, cleaning, deodorisation and care of all suction systems and amalgam separators. Good for all suction systems. Recommended by DentsplySirona, Planmeca, Morita, Belmont & Heka.



■ MD 555 cleaner Special cleaner for suction systems

Foam-free cleaning concentrate for all kinds of suction systems and amalgam separators. Recommended by DentsplySirona, Planmeca & Heka.



■ OroCup Care system

User-friendly, closed care system for the disinfection and cleaning of suction systems and amalgam separators. Use with Orotol Plus & MD555.