

# Ulticlean Prophylaxis Product Information

Prophylaxe



Ulticlean Prophylaxis Powder  
**STANDARD**

Ulticlean Prophylaxis Powder  
**PERIO PLUS**

**Highest quality at the lowest price**

## ULTICLEAN PROPHYLAXIS

### Characteristics

- Ulticlean Prophylaxis PERIO PLUS is a patent pending product for sub-and supragingival treatment with polishing effect (innovation in grain size and crystal structure leads to fast treatment success and omission of final polish time and cost saving)
- Non-cariogenic formulations
- Clinically proven formulations, approved as medical device

### General Information

- Prophylaxis powder for professional dental cleaning and biofilm (plaque) removal. All powders are very fine and evenly grained. This results in selective abrasion of plaque and at the same time is gentle on soft tissue and dental hard tissues.
- Professional powders for sub- and/or supragingival treatment available in different flavours.
- Application areas / Indications

### Application areas / Indications

#### Ulticlean Prophylaxis Standard Powder

The classic for supragingival applications removes biofilm (plaque) and severe discolouration. If necessary, you can use it to prepare the tooth surface for various treatments. Suitable for all commercially available air polishing devices.

#### Ulticlean Prophylaxis PERIO PLUS Powder (patent pending)

The most versatile, non-cariogenic and pleasant-tasting powder that does not require subsequent polishing. Sub- and supragingival (indications above and below the gum line) can be applied to the natural teeth and implants. Suitable for all commercially available air polishing devices.

### Most versatile and efficient treatment!

### MP Class I CE

### Composition


Ulticlean Prophylaxis Powder:  
Sodium hydrogen carbonate based

Ulticlean Prophylaxis PERIO PLUS Powder:  
Tagatose based

### Chemical-Physical Data

Optics:

White, fine and homogeneous powders of defined grain size

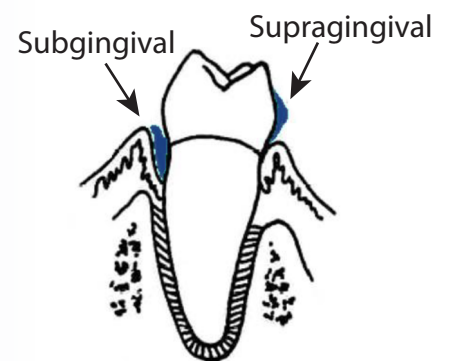

#### Additional indications:

- Cleaning of implant surfaces
- Gentle cleaning in orthodontics (e.g. brackets)
- Cleaning of teeth before bleaching

## ULTICLEAN PROPHYLAXIS

### Variations

The Ulticlean prophylaxis powders are available in the following flavours:

- Mint ( 9796715)
- Lemon (9796714)

Ulticlean Prophylaxis PERIO PLUS (9796716) Powder:

- Neutral

### Instructions for use

Depending on the indication, use per polishing unit manufacturer's instructions. Store powder in a dust-free and dry place and do not mix with other powders.

#### Ulticlean Prophylaxis powders :

Observe the instructions for use and maximum fill level of the device manufacturer. Not suitable for the removal of plaque or calculus in gingival pockets. Also do not direct the air polishing system at the gums or other soft tissue. Exposed dentin and enamel caries must not be irradiated, as higher abrasive values are to be expected here. Composite, compomer, amalgam and gold fillings must also not be irradiated, as roughening may occur. Patients suffering from chronic bronchitis or asthma must never be treated with an air polishing unit. Patients on a low sodium diet (e.g. high blood pressure) should not be treated with the powder as it contains sodium hydrogen carbonate. Patients with a suspected allergy to flavouring substances should not be treated with the powder. Patient and operator must wear eye protection. The operator must wear a mask and gloves. Use a high-performance suction device. Apply Vaseline to the patient's lips. For dental use only. Keep container closed. Shake well before using!

### Certificate

Our products are scientifically examined and tested using the latest methods (e.g., grain size distribution measurements)

### Optimum label dimension

300g: 205 x 80 (Width x Height)

### Available containers

Ulticlean Prophylaxis Powder:  
300g white bottle with adapter closure

Ulticlean Prophylaxis Powder PERIO PLUS:  
120g white bottle with adapter closure



## ULTICLEAN PROPHYLAXIS

### Instructions for use

#### Ulticlean Prophylaxis PERIO PLUS :

Please note the instructions for use and the maximum filling height of the polishing device.

Subgingival application:

- Removal of periodontal biofilms.
- Also suitable for implant cleaning and periodontal therapy

Additional supragingival application at PERIO PLUS:

- Removal of supragingival biofilms and discolouration around the sulcus.
- Suitable for removing discolouration in sensitive areas. Final polishing of the teeth is not necessary and it is suitable for prophylaxis treatment as well. Do not direct the powder stream at fillings, crowns and bridges as these dental restorations could get damaged.

Never treat patients suffering from chronic bronchitis or asthma with an air polishing device. Do not use if sensitive to glycine. The patient and operator must wear eye protection. The operator must wear a mask and gloves. Use a high performance suction device. Apply Vaseline to the patient's lips. Do not direct the powder stream at fillings, crowns and bridges as these dental restorations could get damaged. Only for dental use. Keep container closed. Shake well before using!



### Contact

Henry Schein Australia  
Building 3, Level 6, 189  
O'Riordan Street, Mascot, NSW  
2020

- ☎ 1300 65 88 22
- ✉ [customer.care@henryschein.com.au](mailto:customer.care@henryschein.com.au)
- 🌐 [www.henryschein.com.au](http://www.henryschein.com.au)

