

dentisan[®]

INNOVATION EDUCATION COMPLIANCE

Bioclear[®] Daily

DISINFECTANT
FOR DENTAL UNIT WATER LINES

**Water maintenance solution
for continuous use in
dental unit water lines**

Bioclear Daily is a disinfectant for continuous use in dental unit water lines. It reduces planktonic bacteria counts in water flowing through the unit, minimising the potential for biofilm formation.

Directions for Use

Before first using Bioclear Daily, we recommend a purge clean using a suitable biofilm remover to ensure that the water line system is free from biofilm. Quarterly monitoring is recommended to ensure continued freedom from biofilm contamination.

For use in dental unit water line feed bottles, dilute Bioclear Daily to 1% v/v using distilled or RO water. Refer to the dilution table below. Prepared solutions can be stored in clean, sealed, clearly labelled plastic containers for a maximum of 10 days. When preparing solutions, never pour unused product back into the bottle.

Volume of 1% solution required	Distilled or RO Water (ml)	Bioclear Daily (ml)
750ml	742.5ml	7.5ml
2000ml	1980ml	20ml
5000ml	4950ml	50ml

For use in dental units with compatible built-in automated dosing systems, add Bioclear Daily directly to the disinfectant container. Ensure the dosing equipment is set to deliver a concentration of 1%. Consult your engineer where necessary and always follow the device manufacturers instructions.

Do not mix with other products.
Please refer to product safety datasheet before use.
Use only as directed.

Store upright in a cool, dry place out of direct sunlight.
Store only in original bottle with original cap.
Mop up any spills immediately using a damp cloth.

Health & Safety

Bioclear Daily is not classified as hazardous according to Regulation (EC) 1272/2008, however, we recommend wearing gloves and eye protection when handling the undiluted product. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority for the member state in which the user and/or patient is established



Quadralene Ltd, Bateman Street, Derby DE23 8JL

t: +44(0)1332 268526

w: www.dentisan.co.uk e: info@dentisan.co.uk

REF

DS024

rev 03 - 20/01/20



1Le