

Surface Disinfection

Spray • Wipe • Spray

CLEAN WITH CAVICIDE BEFORE DISINFECTING

1. Spray **CaviCide**[®] directly onto surface to be cleaned.



2. Wipe with paper towel to remove debris and bioburden.



3. Discard used paper towel.



DISINFECT AFTER CLEANING

1. Spray **CaviCide** directly onto a pre-cleaned surface, thoroughly wetting area. Allow three minutes for maximum disinfection.



Follow product label instructions for appropriate contact times.

Wipe • Discard • Wipe

CLEAN WITH CAVIWIPES BEFORE DISINFECTING

1. Unfold first **CaviWipes**[®] towelette.



2. Wipe surface or items with towelette to remove debris and bioburden.



3. Discard used towelette.



DISINFECT AFTER CLEANING

1. Wipe pre-cleaned surface with second **CaviWipes** towelette to disinfect.



Follow product label instructions for appropriate contact times.

Use **CaviCide** or **CaviWipes** to clean, disinfect and decontaminate hard, non-porous surfaces or items to prevent cross contamination, such as:

- Procedure lights
- Dental operatory and laboratory counter tops/cabinets
- Telephones and office machines

Use **CaviCide** for:

- Potentially contaminated non-critical instruments or equipment
- Dental items prior to sterilization

Wear appropriate personal protective equipment when handling chemicals and contaminated items.

| PART NO. | PRODUCT DESCRIPTION | PACKAGING |
|----------|--------------------------------------|--------------------|
| 13-1024 | CaviCide 24 ounce spray | 12 bottles/case |
| 13-1000 | CaviCide gallon | 4 bottles/case |
| 13-1025 | CaviCide 2.5 gallon | 2 bottles/case |
| 13-1008 | CaviCide 8 ounce spray | 12 bottles/case |
| 13-1100 | CaviWipes – 160 wipes/can | 12 canisters/case |
| 13-1150 | CaviWipes XL – 66 wipes/can | 12 canisters/case |
| 13-1155 | CaviWipes XL – individual towelettes | 6 boxes of 50/case |
| 13-1224 | CaviWipes FlatPack – 45 wipes/pack | 20 packs/case |



For more information, contact us at **800.841.1428** or visit **KerrTotalCare.com**



77-1040-2