

**THE POWER, SPEED,
AND EFFICIENCY
YOU'VE BEEN WAITING FOR.**



Maxima Advanced MA210
ULTRASONIC CLEANER
BACKED BY A 3-YEAR WARRANTY!

 **HENRY SCHEIN®**
BRAND PRODUCTS

Engineered for powerful cleaning in as little as five minutes*

Now introducing Henry Schein's newest Ultrasonic Cleaner: Maxima Advanced MA210. Our quietest unit to date! With uncompromising performance and design, our compact unit has the **POWER**, **SPEED**, and **EFFICIENCY** you'll want for your practice.



BACKED BY THREE-YEAR WARRANTY



**INDUSTRY
LEADING
CLEAN TIME**



**POWERFUL
CLEANING
TECHNOLOGY**



**QUIET
OPERATION
(69dB)**



**COMPACT SIZE FITS
COMFORTABLY IN
TIGHT SPACES**



**BUILT TO LAST
WITH DURABLE
ALUMINUM**

*Before starting with routine cleaning, test runs should be conducted to establish proper cleaning time. With a standard batch of instruments, run cleaning at 5 minutes then check the condition of the instruments. Add more time until satisfactory result is achieved. Repeat the test a couple of more times to establish a reliable, proper and standard time. Typical cleaning time is 5 minutes for one layer of loose instruments or cassette.



FRONT VIEW



BACK VIEW

DEGAS DIAL AND TIMER

to prepare for maximum ultrasonic cleaning performance (60, 120 or 180 sec)

EASY-TO-READ

display with bright numbers and icons

START & STOP DIAL/SET TIME

for reliable and accurate timer settings (5, 7:30, 10, 12:30, 15, 20 mins.)

QUICK ACCESS

to a large 2.1 gallon capacity tank for easy loading and removal of instruments

SMOOTH CONTOURED EDGES

designed with a sleek style in mind

BUILT-TO-LAST

durable aluminum panelling

COMPACT SIZE

9 inch height fits comfortably under wall cabinets in your reprocessing area

EASY DRAINAGE ACCESS

located on back of the unit so it can be placed where it works best in your reprocessing area

HANDY LID STORAGE

located on rear panel to keep your workspace open

ALUMINUM HANDLES

for easy grip while transporting machine

Maxima Advanced MA210

Ultrasonic Cleaner HS-570-3794

Large 2.1 gallon tank fits
2 large cassettes or up
to 4 small cassettes

Three stack transducer
technology for power-
ful cleaning



INCLUDED IN BOX: Ultrasonic Cleaning Unit | Stainless Steel Basket Drainage Tube (39" Long) | Power Cord | Instructions for Use | Quickstart Guide

Features

Tank Capacity	2.1 Gallon
Tank Size	13.0" x 9.4" x 4.6"
Longest Item Fits Inside Tank	14.4"
Timer Settings	5 min 7.5 min 10 min 12.5 min 15 min 20 min

Degas Timer Settings	60 sec 120 sec 180 sec
Ultrasonic Frequency	48 KHz ± 3KHz
Net Weight	31.5lb
Unit Size	18.3" x 13.8" x 9.0"

Ready to order?

Product Code	Product Name
HS-570-3794	Maxima Advanced Ultrasonic Cleaner MA210
Spare Parts	
HS-570-3824	Power Cord
HS-570-3805	Drain Hose with Elbow
HS-570-3806	Stainless Steel Basket
HS-570-3803	Large Cleaning Chamber
HS-570-3804	Small Cleaning Chamber
HS-570-3802	300mL Beakers Set of 2



HENRY SCHEIN®
BRAND PRODUCTS

Shop Exclusively at:

HENRYSCHEIN.COM.AU

Building 3, Level 6/189 O'Riordan St, Mascot NSW 2020
Customer.Care@HenrySchein.com.au | 1300 658 822