



## **FINALE SOLVENT**

### DIRECTIONS FOR USE AND SAFTEY INSTRUCTIONS

## POISON

#### KEEP OUT OF REACH OF CHILDREN READ SAFETY DIRECTIONS BEFORE OPENING Finale SOLVENT



# Finishing solvent for smoothing trimmed edges on mouthguards, impression trays and splints

#### Contains: Trichloroethylene 95% v/v

**DIRECTIONS FOR USE:** Moisten a cotton bud with Finale Solvent and rub along marginal edge to be smoothed. Continue rubbing with wetted cotton bud until desired finish is obtained. Thoroughly wash mouthguard/impression tray to make sure no Finale residue remains.

**SAFTEY DIRECTIONS:** Vapour is harmful to health on long exposure. Avoid contact with eyes and skin. Wear a protective mask and gloves when mixing or using. Avoid breathing the vapour. Use only in a well ventilated area. Do not Swallow.

**FIRST AID:** For advice contact the Poisons Information Centre (Australia 131 126, New Zealand 0800 764 766) or a doctor. If swallowed, do NOT induce vomiting. If in eyes, hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by the **Poisons Information Centre (Australia 131 126, New Zealand 0800 764 766)**; a doctor, or for at least 15 minutes.

If skin or hair contact occurs, remove contaminated clothing and flush skin and hair with running water.

**Ingredients:** Trichloroethylene 95% v/v

Storage: Store below 30°C. Keep lid tightly closed