



HENRY SCHEIN®

B R A N D

Alcohol Wipes

Isopropyl Alcohol 70%

Henry Schein Alcohol Wipes are hospital grade disinfectant with rapid bactericidal activity. To be used on pre-cleaned surfaces in healthcare environments. Suitable for use on hard non porous surfaces. Not intended for use on medical devices, therapeutic products or on skin.

Features and Benefits

- Hospital Grade hard surface disinfectant
- Bactericidal
- Streak free formulation

Directions for use

First clean the surface to remove all contaminants (use Henry Schein Neutral Detergent Wipe). Once the surface has been cleaned, use the Alcohol Wipe to disinfect and wipe down leaving the surface visibly wet. Allow to dry at ambient temperature. Refer to packaging for detailed information.

Technical Data

- Solution:** Isopropyl Alcohol 70% v/v, Purified water 30% v/v
- Activity:** Tested under dirty conditions. Pseudomonas aeruginosa; Escherichia coli; Staphylococcus aureus; Proteus vulgaris, Salmonella choleraesuis
- Wipes:** Synthetic disposable wipe
- Storage:** Store below 25°C in a cool, dry place.
- Safety:** Flammable, refer to SDS
- Packaging:** 220 wipes in a cannister (HS-5722624). Single use wipes.

Formulated and produced in Australia.



V1 DEC20