




A use film is available here: www.youtube.com

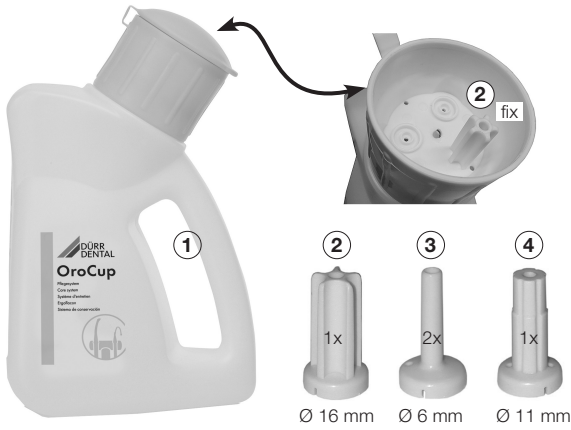

Scope of delivery

- ① OroCup, Order number 0780-350-00
- ② Ø 16 mm cartridge (for large suction hoses), 2x (1x hose, 1x fix)
- ③ Ø 6 mm cartridge (for saliva ejector), 2x
- ④ Ø 11 mm cartridge (other), 1x

Disposable materials

Orotol® plus Disinfection of suction systems.....CDS110P6150
Liquid concentrate

Orotol® ultra Disinfection of suction systems.....CDS120A6750
Powder concentrate

MD 555 cleaner Special detergent for suction systems...CCS555C6150
foam-free concentrate for dental suction units and waste water connections



Product description

The OroCup care system is a closed dosing system for the simple preparation and suction of disinfectants and special cleaners.

Use OroCup to clean and disinfect the suction system and its components and the spittoon.

1 L of ready-to-use solution is required for 1 treatment unit.

OroCup is suitable for conventional suction hoses of various diameters.

The OroCup cover contains 1 fixed cartridge for Ø 16 mm. Two further cartridges can be selected and used depending on the diameter.

Depending on requirements, you can connect 1 - 3 suction hoses at the same time; non-used connections have no influence on the function.

DÜRR DENTAL SE

Höpfheimer Str. 17
74321 Bietigheim-Bissingen, Germany

OPERATION

3. Selecting and inserting cartridges

- Select and insert the cartridge in accordance with the diameter of the suction hose (e.g. Ø 16 mm for larger suction hoses, Ø 6 mm for saliva ejectors), see example.
- Unused connections do not exert an influence on the function.


4. Cleaning and disinfecting the suction unit

Use at the end of all treatment, or twice a day if contamination levels are higher (e.g. afternoons and evening or as required).

- Wear personal protective equipment.


4.1 Pre-cleaning with water

- Suck out 2 l of water.

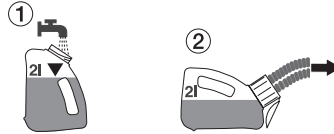


4.2 Place in the OroCup

- Insert 1 or 2 litres of ready-to-use solution as required. Comply with manufacturer information.

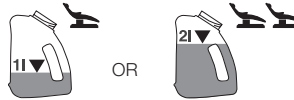

- Close the OroCup cover and shake the OroCup.

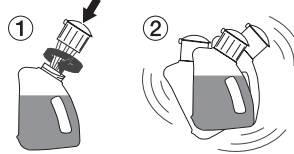


4.3 Position the OroCup and engage suction

- Open the cover of the OroCup.
- Place on the disinfected suction handpiece and remove the ready-to-use solution.

Up to 2 litres of ready-to-use solution can be poured into the OroCup.

Position the OroCup as follows to remove the ready-to-use solution:

- Place in an upright position when sucking out the 1. L (for the 1 treatment unit).
- Place in an horizontal position when sucking out the 2. L (for the 2 treatment unit).

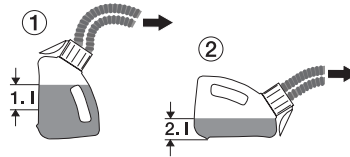

4.4 Cleaning and disinfecting the spittoon

- Use min. 250 ml ready-to-use solution.



4.5 Final rinse

- Suck off 2 K water after the reaction time see 2.1 Pre-cleaning with water.

