

SCANX BARRIER ENVELOPES

THE IMPORTANCE OF MANUFACTURER APPROVED COVERS



- Barrier Envelopes by Air Techniques minimize cross-contamination and infection of intraoral PSPs
- Patented, Easy-Open Barrier Envelopes
- Reduces material costs by extending the service life of the plates
- Does not compromise image quality
- Ensures safe use of the system

Thank you for your purchase and use of the Air Techniques ScanX PSP Unit. The ScanX Barrier Envelopes are the manufacturer approved accessory to accompany your Air Techniques ScanX PSP Unit.

The ScanX Barrier Envelopes were created in order to **ensure the optimal clinical results, safe use of the system and long service life of the imaging plates.**



Figure 7a
Insert the PSP.



Figure 7b
Peel off the adhesive strip.



Figure 7c
Seal the Barrier Envelope.

Air Techniques have created easy-open Barrier Envelopes with the following benefits in mind:

- Protection from contamination provided by the envelope allows PSP sensors to be reused
- Black opaque envelope back prevents ambient light exposure resulting in distorted images
- Top Loading
- Quick Seal
- Easy to tear open
- Our innovative Y-notch design allows you to effortlessly split the envelope open in a direction that reduces cross-contamination

SCANX BARRIER ENVELOPES

THE IMPORTANCE OF MANUFACTURER APPROVED COVERS



Compatible with the ScanX Duo



The manufacturers have perfectly coordinated their image plates and protective covers.

Always follow the manufacturer's instructions when ordering protective covers. This ensures that the highest quality levels are maintained for the diagnostic results, but it also **reduces material costs by extending the service life of the plates.**

Always use a ScanX Barrier Envelope for each Intraoral plate. Exercise care in handling the plate so as not to scratch the sensitive surface or nick the edges.



AIR-73248-0	BARRIER ENVELOPES	SIZE 0 (100)
AIR-73248-1	BARRIER ENVELOPES	SIZE 1 (100)
AIR-73248-2	BARRIER ENVELOPES	SIZE 2 (300)
AIR-73248-3	BARRIER ENVELOPES	SIZE 3 (100)
AIR-73248-4	BARRIER ENVELOPES	SIZE 4 (50)



To order the ScanX Barrier Envelopes, please visit, www.henryschein.com.au or contact Henry Schein on 1300 65 88 22