

Product information

FD 360 cleaning and care of vinyl upholstery



FD 360 at a glance

- Cleaning fluid for deep cleaning and care of synthetic leather surfaces
- Intensive cleaning action – removes stains of all kinds, spots and discolourations
- Excellent material compatibility – suitable for synthetic leather coverings
- Environmentally safe, propellant-free spray bottle
- Set with 500-mL spray bottle, 9 special sponges and 1 special cloth

Properties

FD 360 from Dürr System Hygiene cleans and preserves all synthetic leather upholstery, significantly maintaining the value of treatment chairs and other furnishings with synthetic leather coverings. Simultaneous use of FD 360 and the special cloth cleans and cares for synthetic leather surfaces. With the special sponge, FD 360 quickly and deeply removes even stubborn stains, soiling and discolourations caused, for example, by non-colourfast clothing. FD 360 provides for deep and intensive, residue-free cleaning and care with a silky shine.

Product composition

The FD 360 mechanism of action is based on a combination of special surfactants, silicone compounds and avocado oil-based care components.

Directions for use

Apply FD 360 cleaning fluid undiluted. Spray FD 360 on the surface to be cleaned and immediately wipe with the special cloth or scrub with the special sponge for intensive cleaning with FD 360. Using the sponge enables more thorough, deeper cleaning. Combining FD 360 with the special sponge removes even stubborn stains and discolourations from upholstery. Then remove any excess FD 360 by wiping the surface with a dry cloth. Use FD 360 daily, as needed.

Environmental impact

The packaging is made of polyethylene and polypropylene and can thus be recycled or incinerated. For recycling, flush bottles with water.

For further information see safety data sheet.

Physical data

Appearance: milky, white liquid

Density: $D = 0.99 \pm 0.05 \text{ g/cm}^3$ (20°C)

pH: 3.6 ± 0.5

Shelf life

3 years

Product information leaflet

FD 360 cleaning and care of vinyl upholstery

Package size

Set with 500-mL spray bottle, 9 special sponges and 1 special cloth in the box.

Storage

Store product at between 5°C and room temperature.

General instructions

FD 360 is safe for use on all synthetic leather surfaces. In the case of other surfaces, such as leather, test for material compatibility in an inconspicuous place. Follow the manufacturer's instructions for the treatment unit. The special sponges must not be used on leather. Do not mix FD 360 with other products. Changes in product colour/odour may occur, especially when stored in sunlight. However, these changes in colour and/or odour have no impact on the cleaning efficacy of the product. Use exclusively by qualified personnel. All serious incidents that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

Hazard warnings

FD 360 is classified in accordance with the CLP Regulation and is not required to be labelled.



Sales

Dürr Dental SE
Höpfigheimer Str. 17
74321 Bietigheim-Bissingen
Germany
www.duerrdental.com
info@duerrdental.com

Manufacturer

orochemie GmbH + Co. KG
Max-Planck-Straße 27
70806 Kornwestheim
Germany
www.orochemie.de
info@orochemie.de