










Pro-Matrix is my preferred matrix system due to ease of use, versatility and excellent clinical outcome.

#### Ash Rayarel BDS MFGDP(UK) Msc.

Ash is the Principal Dentist and owner of Ash Dental Practice and Ash Aesthetics.

### **Directions for Use**

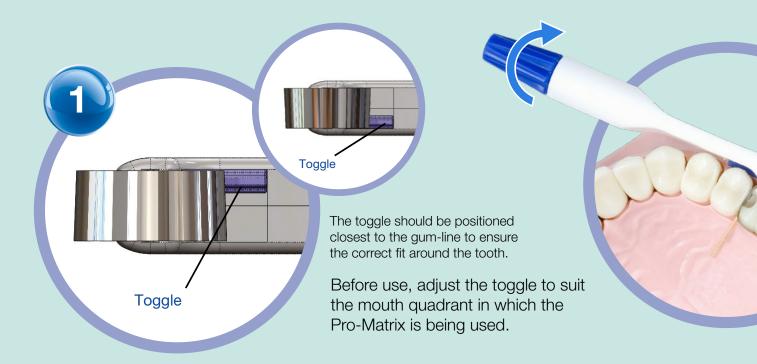



To remove the band, first loosen it by turning the handle 2-3 turns anti-clockwise.



Perform restoration using normal clinical procedure.

# 2

Fit the band around the tooth and tighten by turning the thumbscrew clockwise. Do not over tighten. For larger teeth, unscrew the band anti-clockwise.



Apply wedges and shape the band using a suitable burnishing tool if required.

# Master the art of precision with Pro-Matrix®

Pro-Matrix<sup>®</sup> is a single-use matrix solution for amalgam and composite restorations for use in every class. Available as a standard matrix band Pro-Matrix<sup>®</sup> and Pro-Matrix<sup>®</sup> Curve.

- Single Product System
   no assembly or application tools required
- Circumferential Band can be used for MO/OD/MOD/ Cusp replacements
- High-tension mechanism for better compacting
- Easy contouring to tooth shape
- Narrow front section better visibility and patient comfort
- Suitable for use with all restorative materials
- Suitable for use with wedges and rubber dams
- Fully disposable

# Pro-Matrix Curve has the additional benefits of

- Pre-contoured to tooth shape
- Longer band for larger teeth



# What is the difference between the four colours?

**Green:** Pro-Matrix Narrow

(4.5mm) for pre-molars and children **Blue:** Pro-Matrix Wide (6mm) for molars

**Yellow:** Pro-Matrix Curve Narrow (4.5mm) for pre-molars and children

Orange: Pro-Matrix Curve Wide

(6mm) for molars

## What is the advantage of Pro-Matrix Curve?

The bands are pre-contoured for more natural looking restorations and more predictable contact points.

#### Can it be used with wedges?

Yes. Any wedge can be used.

#### How thick is the Pro-Matrix band?

0.04mm

#### Can it be autoclaved?

No. This product is a single-use disposable item.



#### Is it delivered sterile?

No. It is clean, equivalent to other types of matrix systems. Each Pro-Matrix is individually packed.

# Can it be used with both composite and amalgam?

Yes. Pro-Matrix is suitable for both due to its high-tension mechanism allowing for compacting of amalgam.

# Can more than one Pro-Matrix be used next to each other?

Yes. They stack tightly for patient comfort and excellent visibility.



























#### Young Innovations Europe GmbH

Mittermaierstr. 31 | D-69115 Heidelberg | Germany DE: +49 (0) 6221 4345442 I info@ydnt.eu

GB: +353 (58) 45 966 I info@ydnt.eu

Visit us for more information: www.promatrixdental.eu

AS-GB-PM/0821