SAME-DAY DENTISTRY. UNPARALLELED PERFECTION.

The 4-axis milling machine for wet machining for restorations in mere minutes.





TOMORROW'S LEADING TECHNOLOGY. AVAILABLE TODAY.

The Z4: More than state of the art. Groundbreaking.

The digital workflow makes it happen: you and your patients can benefit from more relaxed treatments with first-class restorations in just one session. The Z4 is an investment that will pay off for you: high-quality

restorations with maximum independence. The number of machinable block materials constantly increases as well as the number of scanners and CAD software packages that are validated with the Z4.





Toolless block clamping in just two seconds. It couldn't be easier.

Perfect results. Mill and grind reliably with the Z4 in Ultra-HD.

EITHER WAY: THE Z4 FITS PERFECTLY INTO YOUR DIGITAL WORKFLOW.

Integrated workflow with 3Shape, exocad

One user interface and everything in your practice: scanning, designing, milling.



Unlimited possibilities with open STL workflow

Design in your practice or your favourite laboratory for maximum indication and material freedom.



University of Washington



Study confirms: Z4 drills screw-access channels in highest quality

A scientific study at the University of Washington proves that using the Z4, users can drill screw-access channels for hybrid implant restorations into ceramic blocks with no significant difference in flexural strength compared to factory-fabricated materials, known as "meso" blocks. These blocks are significantly more expensive, and the laboratory design method requires a more sophisticated CAD process. Thus, processing standard blocks with the Z4 means less cost for material, less material inventory necessary and easier nesting.

Jack M. Keesler, DDS, MSD. Effect of milling screw-access channels on flexural strength of CAD/CAM ceramic materials. MSD Master's Thesis, University of Washington, 2019.

FEATURES AND BENEFITS? LOTS OF THEM!



Highest precision

- Milling and grinding in Ultra HD
- Proven industrial quality
- 3 µm repetition accuracy



Fastest production

- Restorations in under 10 minutes
- 2-second block insertion
- 100,000 rpm electrical high frequency spindle



Absolute independence

- Around 40 block materials from a great variety of manufacturers
- 800+ prefab titanium abutment blanks from various manufacturers
- Validated with all established scanners and design software
- Fully integrated workflow with TRIOS Design Studio (3Shape) and exocad ChairsideCAD*
- Integrated PC with touch screen and Wi-Fi – no laptop necessary
- Built-in compressed air no compressor needed




Highly economical


- PUREWATER: no grinding additives except for titanium processing
- Mill screw-access channels, to save costs for 'meso' blocks
- Automatic changer for 6 tools
- Self-opening working chamber door and drawer
- Easy to learn, easy to operate
- 24 months warranty
- Very easy operation via DENTAL-CAM software with DIRECTMILL Technology – included in scope of delivery and without license fees
- Tool starter set included
- * Material and indication availability may vary by CAD provider. Full range of indications and materials available in STL workflow.


MATERIAL, MANUFACTURER, INDICATION. ENJOY THE FREEDOM OF CHOICE.

All common block materials up to 45 mm length and even abutments Composites Plastics | Wax Glass ceramics Zirconia Titanium CoCr

High-precision milling and grinding for all common indications				
Crown Bridge	Inlay Onlay	Abutment	Telescopic crown	Model plate
Model cast	Occlusal splint	Model tooth	Implant bar	Veneer
Drilling template	Denture	Secondary crown	Screw- retained bridge	Protrusion splint

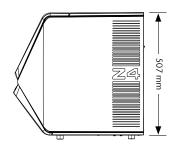

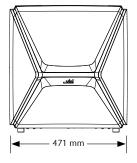

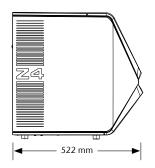


TECHNICAL DATA.

GENERAL			
Fields of application	Wet machining		
Materials	Glass ceramics, titanium, zirconia, composites, plastic materials - Blocks up to 45 × 20 × 20 mm		
Indications	Crowns, bridges, fully anatomical crowns and bridges, inlays, onlays, abutments, veneers, table tops		
BASE SYSTEM			
Construction	Machine bed made of solid cast aluminum body		
Housing	Thick-walled TSG injection moulding, white high-gloss lacquer finish with workspace flap and combined drawer for cooling liquid/tool		
Number of axes	inserts 4		
Linear axes	Precision ball screws, rolled version · motors with resolution < 1 μm · ground precision steel guides · repetition accuracy ± 0.003 mm		
X-/Y-/Z-axis Rotary axis	Rotary axis with high concentricity · Angle of rotation: 200°		
A-axis	Rotary axis with high concentricity. Angle of rotation, 200		
Control unit	4-axis simultaneous control electronics with continuous path progression and dynamic pre-calculation · hardware-based real-time operating system with standardized command set · FPGA-integrated processor · updateable hardware · real-time path calculation via dedicated hardware engines in the FPGA · four-quadrant control of the motors for particularly smooth running · multiple analogue and digital I/Os for controlling the peripherals · integrated inverter for synchronous and asynchronous motors, electronic gate detection · Ethernet and USB interface		
Compressed air generation	Internal compressed air supply with integrated sound insulation		
Lighting	RGB LED lighting with status display		
Camera system	Integrated in the working chamber for easy remote support and possibility of internal recording		
PC (integrated)	Intel® Atom™ E3950, 4 cores, 1.60 · 2.00 GHz,		
Displan	8 GB RAM · additional HDMI display interface · Wi-Fi		
Display	Capacitive 5-inch touchscreen display fully integrated into the front flap for local operation of the machine.		
SPINDLE			
General	High-frequency spindle, asynchronous with pneumatic tool clamping · sealing air to prevent debris from entering · automatic cone cleaning		
Speed	Up to 100,000 rpm		
Power	Peak power (Pmax): 340 watts ⋅ nominal power (S6): 220 watts ⋅ continuous power (S1): 170 watts		
Bearing	Hybrid ceramic ball bearing · concentricity deviation at inner cone < 2 µm		
Collet	Stainless steel collet with ceramic coating for tools with a shank diameter of 3 mm and max. 35 mm total length		
AUTOMATION			
Tool change	Tool magazine for 6 tools, removable and material-coded · Length measurement and tool monitoring control via precision measuring key		
Workpiece change	The integrated DIRECT BLOCK Technology automatically takes over the clamping and releasing of the block or abutment holder to be machined		
Access to the working chamber	Motorized opening and closing of the work chamber flap, movement parallel to the chassis		
Access to combined drawer	Electric ejector for tool and cooling liquid tank drawer		
PROCESSING MODES			
Wet	Multiple liquid nozzles on the spindle · Integrated cooling liquid (2 litres) with active carbon filter system · flow-sensor for monitoring the liquid supply · PUREWATER: no grinding additives except for titanium processing		
CONNECTION REQUIREMENTS			
Compressed air	No compressed air required		
Power	100 · 240 volts · 50/60 Hz, 750 watts		
Data	10/100/1000 Mbit/s BaseT port (auto-sensing) Ethernet via RJ-45 socket		
ENVIRONMENTAL CONDITIONS			
Operating temperature	Between 10 °C and 35 °C		
Air moisture	Max. 80 % (relative), non-condensing		
APPROVALS	· · · · · · · · · · · · · · · · · · ·		
All models	CE, VDE		
North America model	UL, FCC (according to ANSI/UL 61010-1)		
DIMENSIONS & WEIGHTS	out of accounting to recording to the order of		
Dimensions (W/D/H)	471 × 522 × 507 mm with closed flap and drawer		
Footprint (W/D)	$471 \times 737 \times 608$ mm with open flap and drawer 400×305 mm		
Weight SCOPE OF DELIVERY	66 kg		
	DENITAL CAM astruara included		
CAM Software	DENTAL CAM software included		
Accessories	Abutment holder devices for various systems (optional) Spindle service set · Calibration set incl. micrometer · Brush for nozzle plate · Cleaning brush · Microfibre cloth · Spare fine filter · Active carbon pellets · Tec Powder (3 bags) · Spare wiper for viewing window · Tool magazine inserts (5 pieces) · Torque wrench · 2 Allen wrenches · Drill bit (tool positions) · Measuring pin · Power cable · Ethernet network cable · Carrying aid for transporting the machine		



Alan Jurim, DDSDirector for Digital Dentistry at Touro Dental College, New York

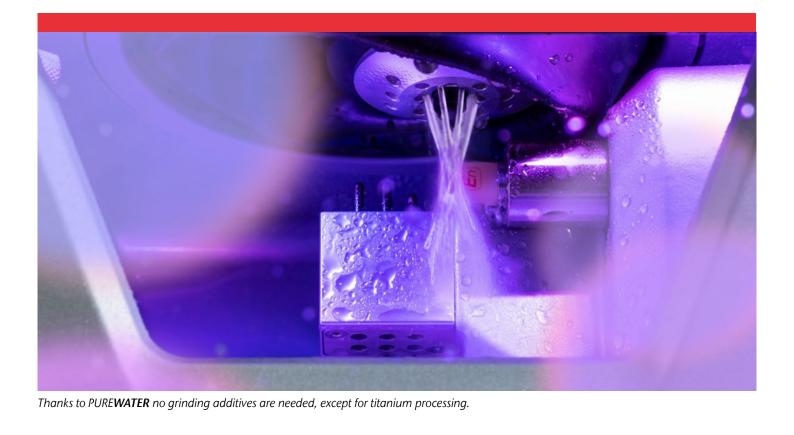



CREATING PERFECTION.

For more than 30 years.

As CAM solution provider, vhf thoroughly develops and produces every single milling machine and the perfectly matching tools and CAM software. Everything from one source. Made in Germany.

Support. A topic close to our hearts.

The service of your machine is important to us: We train our sales partners according to the highest requirements - so you receive first-class support for your Z4.

GET IN TOUCH.

HQ Europe

vhf camfacture AG Lettenstraße 10 72119 Ammerbuch Germany +49 7032 97097 000 info@vhf.de | vhf.de

The Americas

80 Davids Drive, Suite 5 Hauppauge, NY 11788 USA +1 631 524 5252 info@vhf.com | vhf.com

Asia

vhf Trading (Shanghai) Co., Ltd. Room 2902, Building T1, Tianshan SOHO, No. 421 Ziyun Road, Changning District, Shanghai

asia@vhf.de | asia.vhf.de





