### Ask A-dec about...

# **ICXRenew**<sup>™</sup>

#### DENTAL UNIT WATERLINE TREATMENT SOLUTIONS

## **Waterline Shock Treatment**

Fast-acting liquid shock solution removes odour and foul-tasting bacteria in dental unit waterlines. Just mix the two solutions together into an empty water bottle, run through the waterlines, and soak overnight. The blue colour lets you know that the solution is in the waterlines. After completing the shock treatment, flush the waterlines with water and resume your regular use.

#### The Benefits of ICX Renew:

- Penetrates and removes the build-up of odour and foul-tasting microbial contamination bacteria.
- Will not corrode or clog waterlines.
- Blue colour lets you know the solution is hard at work in the waterlines.

ICX Renew – Case (36 applications) 90-1691-00



- Non-corrosive and gentle on equipment.
- Odourless formula.
- Recommended by A-dec, for A-dec equipment.

When used in conjunction with regular testing and adherence to protocols, A-dec ICX and ICX Renew help simplify your waterline maintenance routine.

ICX Renew – Box (9 applications) 90-1691-01



# Unwanted guests in your waterlines?



Turn off delivery system, remove and empty the water bottle.

5



Remove syringe tips, handpieces, detachable motors and couplings, and ultrasonic handpieces from their tubing.



Add the contents of both ICX Renew bottles (solutions 1 & 2) and swirl to let the product coat all surfaces.



Re-attach water bottle and turn delivery system on.



Run solution through the Dental Unit Waterlines (DUWL) until you start to see blue water from the tubing and cuspidor cupfill (where relevant).



Let the product sit in the DUWL for 12-96 hours.



8

Rinse water bottle thoroughly. Refill bottle with water and flush through DUWL until empty.

© 2021 A-dec Inc. All rights reserved.



Add ICX tablet to new water to maintain your Dental Unit Waterlines.

#### When you SHOCK, let ICX RENEW.

For more information on ICX Renew, visit: https://australia.a-dec.com/clean-and-maintain/icx



Watch the video



