Shine reNEW™ Instrument Wipes

Shine reNEW Instrument Wipes clean, polish and protect instruments. Easily removes rust, stains, water deposits and tarnish while polishing dull, discolored metal to keep instruments looking like new.

Safe for use on:

- Stainless Steel • Copper • Brass • Anodized Aluminum

DIRECTIONS FOR USE

Open lid, squeeze the canister and pull lid directly up to remove entire lid from canister. Peel off seal and discard. Locate wipe at the center of the roll, twist to a point and thread through small opening in lid. Replace lid and pull wipe through. Always snap lid securely shut between uses to prevent moisture loss. To remove stains, wearing gloves rub effected area with cloth until stain is no longer visible. For heavily stained spots, rub area with greater pressure. After use, wash and sterilize instrument as per normal instrument processing.

KEEP OUT OF REACH OF CHILDREN.

MSDS Available upon request or at Hu-Friedy.com

SDS Available upon request or at Hu-Friedy.eu

Store in cool, dry place.

For professional use only.

Xi-Irritant

R36: Irritating to eyes.

S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

Eye irritant. Avoid contact with eyes and skin. Wear rubber gloves and safety glasses with side shields. Keep out of reach of children. EYE: Immediately flush with cool water. Remove contact lenses, if applicable, and continue flushing for 20 minutes. Obtain medical attention immediately. SKIN: Brush away excess of dry material. Immediately flush with cool water for 15 minutes. Obtain medical attention if irritation persists. Wash contaminated clothing well before reuse or discard. INHALATION: If symptoms develop move victim to fresh air. If symptoms persist, obtain medical attention. INGESTION: Do not induce vomiting. Never give anything by mouth if victim is unconscious, or is convulsing. Obtain medical attention.

READ MATERIAL SAFETY DATA SHEET BEFORE USING PRODUCT

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LIRE LA FICHE SIGNALÉTIQUE AVANT D'UTILISER CE PRODUIT Hu-Friedy Mfg. Co., LLC