

## Valisafe Class 6 Indicator (200 per Box)

Order Details: **9VAL-CLASS6**

These indicators are calibrated specifically to your cycle requirements. Valisafe Class 6 indicators offer validated security that every tray or item has been effectively processed, thereby providing a direct link between the sterility of the items and the patient.

### When would you use a Class 6 indicator?

- As an optional and supplementary testing protocol on top of IQ/OQ/PQ (See AS4815:2006 7.7.8 & Table 7.1)
- Where on-site technical support is not available to achieve calibration, operational qualification and performance qualification for new sterilizers, temporarily loaned sterilisers, or sterilisers returned following maintenance or repair, additional monitoring with a Class 5 or 6 chemical indicator shall be used in every pack in every load.....” (AS4815:2006 7.3.1.3)
- For existing steam sterilizers where no permanent record of the physical parameters of each cycle is obtained, either by print-out or by direct observation, a biological/enzymatic indicator or a Class 4, 5 or 6 chemical indicator shall be used with each load (AS4815:2006 7.7.4.1)


- Conforms to EN ISO 11140–1 Class 6
- Specifically calibrated to 3.5 min at 134°C cycle
- Class 6 emulating indicator
- Monitors critical parameters – Time – Temperature – Steam
- Clear and abrupt colour change
- Easy to interpret—easy to record
- Non-toxic / Lead free



### Colour change: pink to brown or darker

Before  
Original indicator

After  
After total process

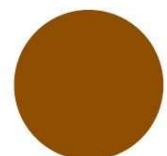

Name:	Class 6 Indicator
Classification:	EN-ISO 11140-1 (Class 6)
Operating temp:	134°C
Time:	3.5 min
Storage conditions:	10°C - 30°C
RH:	30 - 70%
Correct symbols:	In accordance with EN-ISO 11140-1 (Class 6)
Indicator:	Class 6
Disp/single use:	Yes
Coated:	Yes
Non-toxic/Lead free:	Yes, according to EC Regulation 1907/2006
Lot no.	Yes
Prod date/Exp date:	Yes
Operator:	Yes
Pass/fail result:	Yes
Colour change:	Pink to brown or darker

**Shelf Life** - 3 Years from date of manufacture. Typically 2 1/2 years remaining at time of sale



12 Juna Drive, Malaga, WA 6090 PO Box 1810, Malaga, WA 6944

T (08) 9244 4628

orders@sts-group.com.au www.stshealth.com.au