

SciCan

VistaPure AutoFill™

Look, doc: no hands!

VistaPure's patented AutoFill accessory replenishes SciCan STATIM® autoclaves with distilled-quality water automatically—enabling truly hands-free convenience.

VistaPure AutoFill accessory kits work with all VistaPure systems and can be installed easily on any STATIM 2000S/G4 or 5000S/G4 in just a few minutes!

- ✓ No more purchasing and storing distilled water
- ✓ No more cycle-fault errors caused by forgetting to fill reservoirs
- ✓ No more spills while filling autoclaves manually
- ✓ No more staff time spent filling autoclaves = more time with patients




AutoFill for STATIM S3020

VistaPure AutoFill Installation

A 12-foot length of clear polyurethane tubing is already attached to the AutoFill manifold. If the desired distance between the connection to the VistaPure and the autoclave is greater than 12 feet, simply order a longer length of the clear polyurethane tubing, as the distance from the VistaPure to the autoclave is not limited, except by the feed tubing.

Installing AutoFill on a STATIM

The VistaPure AutoFill for STATIM device (S3020) can be installed on any new or older model 2000S/G4 or 5000S/G4 STATIM autoclave as follows:

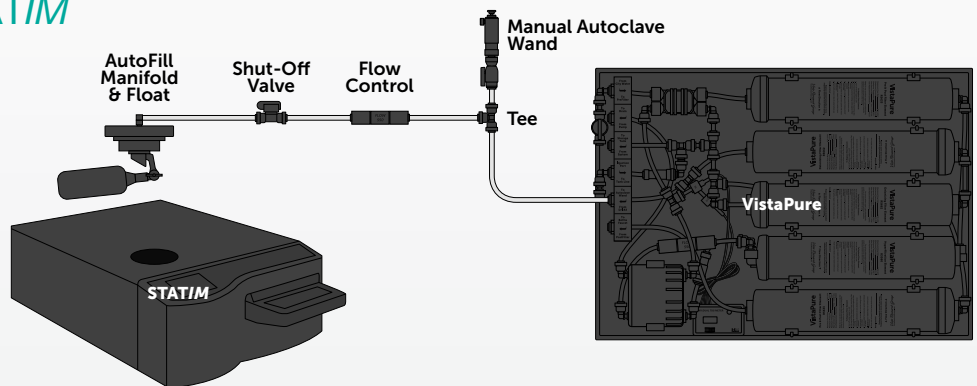

Distributed by:
SciCan Ltd.
1440 Don Mills Road
North York, ON M3B 3P9 / Canada
T +1 416 445 1600
TF +1 800 667 7733
customerservice@scican.com

VistaPure and AutoFill are manufactured by:
Crosstex International, Inc.
1733 W. Parkside Lane
Phoenix, AZ 85027 / USA
TF +1 800 531 2221
VistaResearchGroup.com

www.scican.com

 **COLTENE**

STATIM is a registered trademark of SciCan Ltd. VistaPure and AutoFill are trademarks of Crosstex International, Inc., Phoenix, AZ.