# Impregum<sup>™</sup>

Soft Polyether Impression Material






# This material makes it easier to capture accurate impressions the first time, reducing retakes and adjustments.

Many doctors use both polyether and VPS impression materials. But when the situation calls for capturing fine detail — even in a wet environment — then Impregum™ Soft Polyether Impression Material is the one to reach for. Its characteristic profile helps you capture accurate detail the first time — which can lead to fewer retakes, remakes and adjustments for you, and a more comfortable restoration for your patients.

The less-rigid, *SOFT* polyether gives you the same great results, but is easier to remove. And, the NEW mint flavor is more pleasing to your patients.

## An ultimately forgiving "snap-set behavior" means less stress for dentists and dental assistants.

Impregum Soft material is *ultimately forgiving*. Snap-set behavior – which is typical for polyether – means the material will not start setting before the working time ends. When it does set, it does so immediately. You can have greater assurance of achieving precise-fitting restorations without distortion, across a wide variety of techniques.

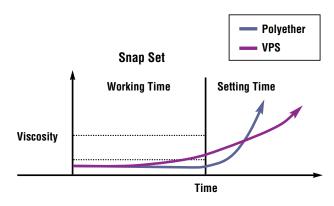

#### **Summary of advantages**

- Better initial hydrophilicity than any VPS – provides void-free impressions in moist conditions
- Excellent flow properties captures fine details for precise impressions
- "Snap-set behavior" of soft polyether is ultra-forgiving
- Less-rigid, soft polyether is easier to remove – great for dual-arch trays
- Quick-Step material cuts working and setting times by at least 33% over Impregum™ Soft Polyether Impression Material

### Outstanding initial hydrophilicity allows excellent detail reproduction in a wet environment.


Polyether is hydrophilic by nature of its chemical makeup. From the time it is mixed, until it sets, moisture in the mouth should not interfere with achieving accurate, void-free impressions!

- Outstanding initial hydrophilicity displaces moisture right from the start of mix when it counts – for void-free impressions!
- Preferred over VPS for excellent detail reproduction in a wet environment!1
- Moisture and saliva will not compromise marginal accuracy or integrity.



Water drop on Aquasil Impression Material prior to set (left). Initially, Aquasil shows hydrophobic behavior. Water drop on Impregum™ Soft Polyether Impression Material prior to set (right).

Source: Mondon M., Ziegler Chr., Reusch B., "Surface Wettability During the Curing Process of Impression Materials," AADR, #618, 2001.

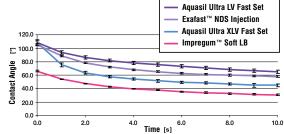

Contact angle measurement on unset material is a method used to determine hydrophilicity. These charted results show why Impregum™ Soft Polyether Impression Material is valuable to dentists for getting precise impressions the first time.

Source: Klettke T., Kuppermann B., Führer C., Richter B., "Hydrophilicity of Precision Impression Materials During Working Time," CED/IADR, Istanbul, 2004, submitted for publication.

#### Impregum<sup>™</sup> Soft Impression Material flows better into tough spots to capture fine detail.

The Shark Fin Test proves that Impregum Soft polyether material flows better into the toughest spots, yet does not slump - capturing more detail than any VPS! In fact, it offers the best flow behavior during the entire working time.









Aquasil Ultra LV Aquasil Ultra XLV

An established method for analyzing flow properties is the Shark Fin Test, developed by impression experts at 3M ESPE. Results shown are at the end of the manufacturer's recommended working time.<sup>2</sup>

#### Shark Fin Test Method



Approximately 10ml of impression material is injected to fill the receptacle of the Shark Fin device. The mixing tip is buried into the receptacle to avoid trapping air.



A fixed mold and 147g weight are placed over the receptacle. The pin is released, allowing the weight to sink slowly into the material. (The 147g weight accurately reflects the pressure applied during clinical placement in the mouth.)






The material is allowed to set. The molds are separated and samples are measured using a caliper accurate to 0.01mm.

The taller the fin height. the better the flow.

## Quick Step material cuts working and setting times by 33%, and helps "lock in" an accurate impression on the first try.

Dentists report Impregum™ Soft Quick Step Polyether Impression Material has enabled them to capture detailed impressions on the first try, and produce restorations that require fewer adjustments. Plus, in their opinions, finished cases look better, and are less stressful to produce.

• Offers maximum working time of 1 minute, and a setting time in the mouth of 3 minutes.

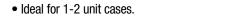

<sup>1</sup> Source: 3M ESPE internal data. International data on file

#### Impregum™ Soft **Polyether Impression Material indications**

- Crowns and bridges
- Inlavs and onlavs
- Functional impressions
- Implant abutments
- Quick-setting materials especially suited for single-unit and two-unit impressions



Photo courtesy of Jorge Perdigao, DMD, and Holmer Meiser, DDS, University of Minnesota.

**11** The hydrophilic properties ensure that my impressions have outstanding marginal accuracy and integrity, despite the presence of moisture and saliva.

> Dr. David DeGrave Green Bay, WI, USA.

<sup>&</sup>lt;sup>2</sup> Photo documentation by 3M ESPE. Data on file.

#### Products designed for the way you like to work ...

For every clinical situation, you'll find an Impregum<sup>™</sup> Soft Polyether Impression Material to suit your technique and procedure. Choose from a wide range of viscosities and set/working times. Tray material now available in both Penta<sup>™</sup> AND handgun delivery.

More proven solutions brought to you by 3M ESPE, the worldwide leader in impression materials.



Choice of two deliveries: **Pentamix™ 3 Automatic Mixing Unit** dispenses completely homogeneous mix at the push of a button. Or convenient **handgun dispensing**.

#### **Ordering Information**

Item #	Product Information	Item #	Product Information
P31738	Impregum™ Penta™ DuoSoft Heavy Body/Light Body Introductory Kit Kit Includes: 1 base paste HB – 300ml; 1 catalyst HB – 60ml; 1 Penta cartridge HB; 10 Penta mixing tips, red; 2 cartridges LB – 50ml; 10 cartridge mixing tips, white; 10 intraoral tips, white; 1 cartridge dispenser; 1 polyether tray adhesive – 17ml	31740	Impregum™ Penta™ H DuoSoft Refill 2 base paste – 300ml; 2 catalyst – 60ml
		31769	Impregum™ Penta™ Soft Quick Step Heavy Body Refill Pack 2 base paste – 300ml; 2 catalyst – 60ml
P31734	Impregum™ Penta™ Soft Medium Body Introductory Kit Kit Includes: 1 base paste MB – 300ml; 1 catalyst MB – 60ml; 1 Penta cartridge MB; 10 Penta mixing tips, red; 1 Penta elastomer syringe; 1 polyether tray adhesive – 17ml	31730	Impregum™ Penta™ Soft Medium Body Refill Pack 2 base paste – 300ml; 2 catalyst – 60ml
		31765	Impregum™ Penta™ Soft Quick Step Medium Body Refill Pack 2 base paste – 300ml; 2 catalyst – 60ml
P31684	Impregum™ Penta™ Medium Body Introductory Kit  Kit Includes: 1 base paste MB – 300ml; 1 catalyst MB – 60ml; 1 Penta cartridge  MB; 10 Penta mixing tips, red; 1 elastomer syringe; 1 polyether tray  adhesive – 17ml	31644	Impregum™ Penta™ Medium Body Refill Pack 2 base paste – 300ml; 2 catalyst – 60ml
		31785	Impregum™ Soft Tray Refill Pack
31786	Impregum™ Soft Tray Introductory Kit  Kit includes: 3 Tray cartridges – 50ml; 1 LB cartridge – 50ml; 10 cartridge mixing tips, purple; 10 cartridge mixing tips, white; 10 intraoral tips, white; 1 polyether tray adhesive – 17ml		4 cartridges – 50ml; 10 cartridge mixing tips, purple
		31788	Impregum™ Soft Quick Step Tray Refill Pack 4 cartridges – 50ml; 10 cartridge mixing tips, purple
		31745	Impregum™ Penta™ L DuoSoft Ligth body wash Refill
31789	Impregum™ Soft Quick Step Tray Introductory Kit		2 base paste – 300ml; 2 catalyst – 60ml
	Kit includes: 3 Tray cartridges – 50ml; 1 LB cartridge – 50ml; 10 cartridge mixing tips, purple; 10 cartridge mixing tips, white; 10 intraoral tips, white; 1 polyether tray adhesive – 17ml	31751	Impregum™ DuoSoft Light Body Refill Pack 4 cartridges – 50ml; 20 cartridge mixing tips, white
		31767	Impregum™ Soft Quick Step Light Body Refill Pack 4 cartridges – 50ml; 20 cartridge mixing tips, white
		71512	50 Penta™ Mixing Tips, Red
		71451	50 Cartridge Polyether Mixing Tips, White
		71454	50 Cartridge Polyether Mixing Tips, Purple





3M Aus Pty Limited

3M ESPE Dental Division Bldg A, 1 Rivett Road North Ryde NSW 2113 Customer service: 1300 363 454 ABN: 90 000 100 096

www.3MESPE.co.nz