



Precision Redefined - Performance Elevated

Discover the Enhanced Version of Our Most Popular File System



The Exceptional EdgeX7 Utopia™

The EdgeX7 Utopia™ combines the advanced technology of the EdgeFile X7® with a stunning gold blade, enhanced by our FireWire Blaze heat treatment. To top it off, we've introduced our Enhanced Fit handle for unparalleled performance.

FireWire™ Blaze Heat Treatment

Years in the making, and worth every moment. With FireWire Blaze, we set out to marry flexibility and cutting efficiency. Experience the exceptional feel, designed to navigate challenging canals with ease and confidence.



**Parabolic
Cross Section**



**Highly
Flexible**



**No Bounce
Back**



**High
Strength**


EDGE X7 UTOPIA™

Enhanced Technology. Remarkably Beautiful. Ultra Secure Shank.



Features

- Available in .04 and .06 Constant Taper until 1mm max diameter -Variable Pitch
- Maximum flute diameter 1mm allows for minimally invasive preparation
- Parabolic Cross Section non cutting tip - Maximises file cutting efficiency
- ISO tip size 17-45
- FireWire Blaze technology
- Sterile Packaging
- Same technique as EdgeFile® X7
- ULTRA-FIT Secure Shank
- Available lengths: 21, 25 & 29 mm

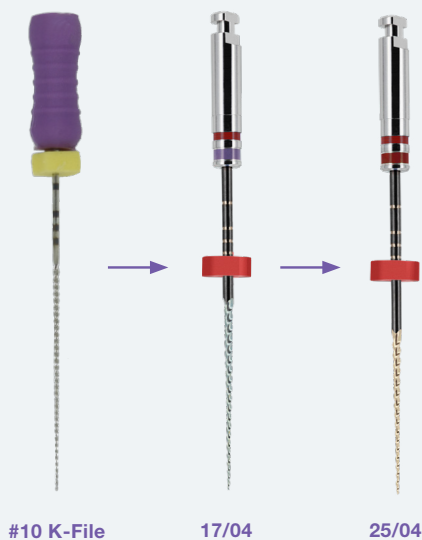

Simplified Technique

1. Create straight line access to canal orifices.
2. Locate canals and explore using stainless steel hand instruments. Minimum size #10K file to working length recommended prior to rotary file use.
3. Irrigate before each hand or rotary file.
4. Use #17/04 file in one or more passes, alternating with small-sized hand files if necessary, until working length is reached. If more coronal flare is desired it can be achieved by incorporating the EdgeGlidePath file used in a brushing motion.
5. Next use #25/04 to working length passively; if instrument has not reached working length use additional shaping instrument #20/04 to working length.
6. If #25/04 reached working length with minimal resistance or if clinician desires a larger apical shape additional instruments can be used (#30, #35, #40, etc).
7. Please note Clockwise rotation - Speed 500rpm. Torque setting 410g/cm.

*This technique is intended to be used as a guide only



To order please contact Henry Schein Customer Service,
your local Henry Schein Relationship Manager, or on Schein On-line.

AU - henryschein.com.au - 1300 769 861
NZ - henryschein.co.nz - 0800 808 855

